**ENGLISH**

For guaranteed performance, use only Welch Alllyn lamps and specula.  
 Conforms with: IEC/UL/CSA/EN 60601-1

**Symbols**

- Attention, Read Operating Manual for Cautions and Instructions for Use.
- Type BF applied part
- Reorder number
- Product Identifier
- Manufacturer & Date of Manufacture (YYYY-MM-DD)
- Professional use only
- Authorized Representative in the European Community
- Global Trade Item Number
- Consult operating instructions
- Batch code

**Warnings/Cautions**

- WARNING:** Lamps may be hot. Allow the lamps to cool before removing.
- WARNING:** To minimize external otoscope housing temperatures, on-time should not exceed 2 minutes and off-time should not be less than 10 minutes.
- CAUTION:** Halogen lamps are pressurized to provide maximum efficiency and illumination. Mistreatment may cause shattering. Protect lamp surface against abrasions and scratches. Use protective eye glasses when handling. Verify power is off when replacing lamp. Dispose of lamp with care.

**Operating Instructions**

**Attaching Oscope Head to power source**  
 (Figure 1)  
 Align the cutouts in the Oscope Head base with lugs on power source. Push in slightly and turn 90°. The Oscope is now ready to use. To remove, reverse this procedure. The Oscope Heads fit all 3.5 v. Welch Alllyn power sources.

**Using Focus**  
 (Figure 2)  
 Use the focus in the Default or Free position.  
**Default Focus:** align the green line on the focusing wheel to the green dot on the side of the oscope.  
**Free Focus:** rotate the focus wheel (A) or (B) to zoom in or out. Zoom in: rotate focus wheel towards longer dashes. Zoom out: rotate focus wheel towards shorter dashes.

**Attaching Specula**  
 (Figure 3)  
 To attach the speculum onto the oscope, twist the speculum clockwise into the TipGrip (C) until it locks.

**Removing Specula**  
 To remove the speculum, turn the TipGrip (C) counterclockwise. Or grab the speculum and twist it counterclockwise.  
**Note:** Oscope only supports Welch Alllyn specula.

**Using Throat Illuminator for REF 23820**  
 (Figure 4)  
 1. Grasp the oscope section with the left hand.  
 2. With the right hand, grasp the throat illuminator's collar at (D) under the silver ring.  
 3. While holding the throat illuminator collar stationary, twist the oscope section counterclockwise at (D) until the oscope head is removed. The throat illuminator is now ready for use.  
 4. To replace oscope section, reverse the steps.  
**Note:** It is important to note that while the throat illuminator section provides an excellent source of general illumination, more than momentary direct skin contact should be avoided due to possible heat build up.

**Lamp Replacement**

(Figure 5)  
**Repairs**  
 Repairs should be performed at the factory.  
**Cleaning and Disinfection**  
 For cleaning the mirror or lenses, use only a clean, dry optical-grade cloth. Do not use alcohol or other chemical solutions.  
 The window may be cleaned with a cotton swab or lens paper dipped in alcohol.  
 The Head housing may be cleaned and disinfected with an appropriate health care low- or intermediate-level cleaner/disinfecting wipe that incorporates either a 1.10 sodium hypochlorite (bleach) solution or isopropyl alcohol as the active disinfecting ingredient.  
 Follow wipe manufacturer's instructions for appropriate use, contact times and applicable warnings and precautions.  
 Do not excessively saturate the Head.  
 Do not immerse Head in any solution.  
 Do not sterilize the Head. After disinfection, inspect for visible signs of deterioration. If evidence of damage or deterioration is present, discontinue use and contact Welch Alllyn or your area representative.

- Accessories:**
- Insufflator bulb REF 23804
  - Lamps REF 06500-U or REF 06500-LED
  - Disposable Specula
    - REF 52432-U, 2.75 mm
    - REF 52434-U, 4.25 mm
  - Reusable Specula
    - REF 24302-U, 2.5 mm
    - REF 24303-U, 3 mm
    - REF 24304-U, 4 mm
    - REF 24305-U, 5 mm
    - REF 24400-U, Set of 4

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Welch Alllyn Limited
Welch Alllyn, Inc.
Skaneateles Falls, NY 13153-4220 U.S.A.